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	Reagent Blank	Standard / Sample
Standard / Sample	---	20 µl
Reagent R1	1000 µl	1000 µl

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Inter Assay – Between Run			
Sample	Mean (g/dl)	SD (g/dl)	CV%
Control Serum 1	5.3	0.1	1.88
Control Serum 2	7.4	0.09	1.21
Control Serum 3	10.5	0.15	1.42

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- Evaluation of the Linearity of Quantitative Measurement Procedures: A Statistical Approach; Approved Guideline. CLSI document EP06-A. Wayne,PA: Clinical and Laboratory Standards Institute; 2003.
- Evaluation of Detection Capability for Clinical Laboratory Measurement Procedures; Approved Guideline-Second Edition. CLSI document EP17-A2. Wayne,PA: Clinical and Laboratory Standards Institute; 2012.
- Interference Testing in Clinical Chemistry; Approved Guideline-Second Edition. CLSI document EP07-A2. Wayne,PA: Clinical and Laboratory Standards Institute; 2005.

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- Tietz NW *Clinical Guide to Laboratory Tests* 3rd Edition
- Henry RJ, et al *Clinical Chemistry Principles and Techniques* Harper & Row 2nd Edition (1974)
- Young DS. *Effects of drugs on clinical laboratory tests*. AACC Press
- Evaluation of Precision of Quantitative Measurement; Approved Guideline-Third Edition. CLSI document EP05-A3. Wayne,PA: Clinical and Laboratory Standards Institute; 2014.
- Evaluation of Stability of In Vitro Diagnostics Reagents; Approved Guideline. CLSI document EP25-A. Wayne,PA: Clinical and Laboratory Standards Institute; 2009.
- Measurement Procedure Comparison and Bias Estimation Using Patient Sample; Approved Guideline-Third Edition. CLSI document EP09-A3. Wayne,PA: Clinical and Laboratory Standards Institute; 2013.

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Adult	6.6-8.7 g/dl
Children up to 6 years	5.6-8.5 g/dl
Neonates	5.3-8.9 g/dl

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